

Carbon Steel Material Cattle Mouth Gag , Mouth Gag for Animal

Our Product Introduction

for more products please visit us on mobilemilkingmachine.com

Basic Information

- Place of Origin: Jiangsu, China(Mainland)
- Brand Name: Chuangpu
- Certification: ISO
- Model Number: HL-Q13
- Minimum Order Quantity: 1pcs
- Price: USD\$10.50/PCS-USD\$11.00/PCS
- Packaging Details: CARTON
- Payment Terms: T/T, L/C, Western Union, MoneyGram
- Supply Ability: 1000PCS PER MONTH



Product Specification

- Features: Simple Operation Easy To Use
- Brand Name: Chuangpu
- Model Number: HL-Q13
- Animal Type Application: Cow/Cattle
- Place Of Original: Jiangsu China(Mainland)
- Is_customized: Yes
- Condition: New
- Item: Mouth Gag For Animal
- Material: Carbon Steel Plating Nickel On Surface
- Highlight: cattle hoof trimming discs, cattle nose ring



More Images



Product Description

Product Model Number: HL-Q13

Product Material: Carbon Steel, plating nickel on surface

Product Specification: Diameter is 12mm, Width is 230mm, Length is 330mm, Net Weight is 0.77Kg.

Model Number	HL-Q13
Diameter	12mm
Width	230mm
Net Weight	0.77kg

Our Product Introduction

产品简介 Product introduction

创普机械 Chuang Pu

创普机械
Chuang Pu



Product name: Cow Mouth Opener

Product number: HL-Q13

Product introduction: The cattle opener is a tool for opening large livestock such as cattle, and is used to assist the injection of drugs, magnetic cages and the like. The whole is made of high-quality metal, the structure design is reasonable, and the damage to the cattle is reduced when the medicine is filled.

产品参数 Product parameters

创普机械 Chuang Pu



Instructions

The opener is a device used by our veterinarians and rural epidemiologists for large livestock such as cattle. Binding method: After the animal is put down, the head limb binding method must be adopted, that is, a rope is inserted into the corner of the animal, a slip knot is pulled over the rope and tightened, and the free end of the rope is pulled back over the front joint. Pull it by the animals. The operator holds the opener in the left hand and inserts the opener iron tube into the animal's mouth. According to the size of the animal, a suitable stomach tube can be selected to insert the esophagus from the middle of the iron tube. After the rubber ball test, the funnel can be used to fill the medicine.



E-mail: sales2@dairymachinery.cc
nancyzxd68@gmail.com

Skype: nancy.zhang62
 Mobile: +86 133 0156 9240 (Wechat/WhatsApp)



E-mail: sales2@dairymachinery.cc
nancyzxd68@gmail.com

Skype: [nancy.zhang62](https://www.skype.com/name/nancy-zhang62)
Mobile: +86 133 0156 9240 (Wechat/WhatsApp)



E-mail: sales2@dairymachinery.cc
nancyzxd68@gmail.com

Skype: [nancy.zhang62](https://www.skype.com/name/nancy-zhang62)
Mobile: +86 133 0156 9240 (Wechat/WhatsApp)



Hailian Packaging Equipment Co.,Ltd



0086 13301569240



sales2@dairymachinery.cc



mobilemilkingmachine.com

Nanzhuang Village, Economic Development Zone, Yangshe Twon, Zhangjiagang City, China