



Dairy cow balling gun for magnet , stainless steel bolus gun with pressing handle

Our Product Introduction

for more products please visit us on mobilemilkingmachine.com

Basic Information

- Place of Origin: Jiangsu, China(Mainland)
- Brand Name: Chuangpu
- Certification: ISO
- Model Number: HL-Q11 HL-Q11A
- Minimum Order Quantity: 1pcs
- Price: USD\$17.00/PCS-USD\$17.50/PCS
- Packaging Details: CARTON
- Payment Terms: T/T, L/C, Western Union, MoneyGram
- Supply Ability: 1000PCS PER MONTH




Product Specification

- Features: Simple Operation Easy To Use
- Brand Name: Chuangpu
- Model Number: HL-Q11 HL-Q11A
- Animal Type Application: Cow/Cattle/Goat
- Place Of Original: Jiangsu China(Mainland)
- Is_customized: Yes
- Condition: New
- Material: Stainless Steel
- Highlight: veterinary medical instruments, cattle hoof trimming discs



More Images

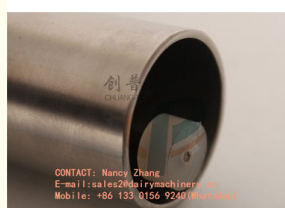

Product Description

Product Model Number: HL-Q11 HL-Q11A

Product Material: Stainless Steel

Model Number	HL-Q11	HL-Q11A
Width	25mm	38mm
Length	650mm	700mm
Net Weight	0.5kg	1.15kg

产品简介

Product introduction

创普机械 Chuang Pu



创普机械
Chuang Pu

Product name: Pill Propellant Gun

Product number: HL-Q11

Product introduction: The pill propellant gun is made of stainless steel material and is used to administer the medicine when the animal is sick. It is a simple and convenient instrument for cattle and sheep. In operation, put the medicine to be fed to the animal into the medicine bucket and then press the awl firmly. After the completion of the filling, the injection gun should be cleaned in time for the next use.

产品参数

Product parameters

创普机械 Chuang Pu

款式一

Style 1



High quality stainless steel

700mm

weight:1.15KG

款式二

Style 2



High quality stainless steel

650mm

weight:0.50KG

Object that can be placed

创普机械
Chuang Pu



Magnetic Trap



Veterinary drug

Instructions

- 1.Put the drug or magnetic cage that needs to be fed into the head of the vat.
- 2.Open the animal' s mouth with an opener and then insert the medicine gun into the animal's mouth.
- 3.After confirming the successful investment, the medicine gun can be taken out.

产品细节

Product details

创普机械 Chuang Pu

创普机械
Chuang Pu

1.

Stainless steel material and durable.



2. Reasonable grip design, comfortable and not hurting the hand.



3. Press-type design, easy to operate.





Hailian Packaging Equipment Co.,Ltd



0086 13301569240



sales2@dairymachinery.cc



mobilemilkingmachine.com

Nanzhuang Village, Economic Development Zone, Yangshe Twon, Zhangjiagang City, China