



Obstetric equipment calf sling for calf puller, nylon midwifery rope , 1m length , 2pcs/set

Our Product Introduction

for more products please visit us on mobilemilkingmachine.com

Basic Information

- Place of Origin: Jiangsu, China(Mainland)
- Brand Name: Chuangpu
- Certification: ISO
- Model Number: HL-Q10C
- Minimum Order Quantity: 1pcs
- Price: USD\$3.00/PCS-USD\$3.50/PCS
- Packaging Details: CARTON
- Payment Terms: T/T, L/C, Western Union, MoneyGram
- Supply Ability: 1000PCS PER MONTH



Product Specification

- Brand Name: Chuangpu
- Model Number: HL-Q10C
- Place Of Original: Jiangsu China(Mainland)
- Is_customized: Yes
- Condition: New
- Material: Nylon
- Net Weight: 10g
- Width: 15mm
- Length: 1000mm
- Highlight: cattle hoof trimming discs, cattle nose ring



More Images



Product Description

Product Model Number: HL-Q10C

Product Name: Obstetric equipment calf sling for calf puller, nylon midwifery rope , 1m length , 2pcs/set


Product Material: Nylon

Product Technical Data: Net Weight 10g, Length 1000mm, Width 15mm.

How to use:

- 1) Disinfect the equipment before use, apply lubricant to the surface.
- 2) Place the opening end of the buttocks of the midwife on the hind leg of the cow that needs to be delivered.
- 3) With the end of the midwife tied to the calf back, and then the other side of the midwife tied to the midwife of the mechanical handle.
- 4) slowly tighten the help after the production of rope, the use of lever principle down the main bar.
Regardless of whether the cow is standing or lying, it is necessary to find the correct position in the process of tightening the midwifery.
- 5) After use, rinse with water, and then sterilization treatment.

Net Weight	10g
Material	Nylon
Width	15mm
Length	1000mm

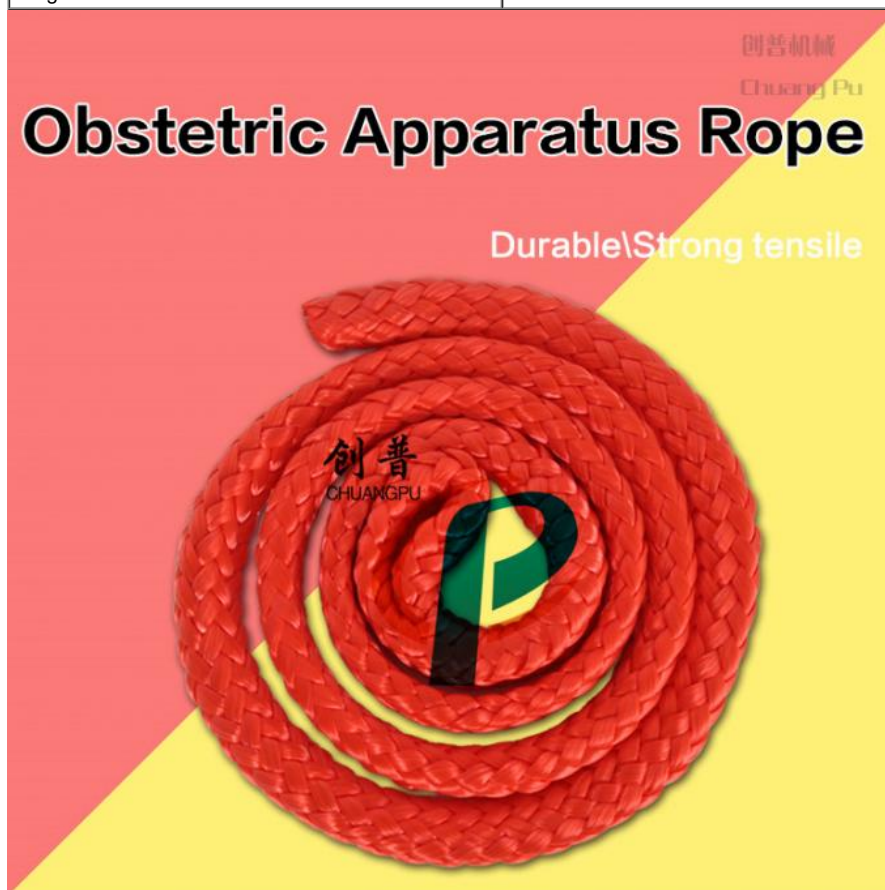



创普机械

Chuang Pu

Product name: Obstetric Apparatus Rope

Product number: HL-Q10D

Product introduction: It is used in dairy cows for the production of yak, nylon material, strong and durable, strong tensile and not easy to be infected. It can be reused, soak it in clear solution after use.



创普机械

Chuang Pu

Length: 1M



Friendly reminder: normal packaging is a package of two, one can also be sold separately.

one can also be sold separately.

Directions for use

The obstetric apparatus rope is used as a tool for the dairy cow to use with the obstetric apparatus machine. When using it, it is necessary to tie the two auxiliary ropes to the yak' s leg, and the other end of the rope is tied to the obstetric apparatus. The power of the squid will smooth out the yak and minimize damage to cows and calves.

产品细节

Product details

创普机械 Chuang Pu



1. Nylon material, strong and durable, strong tensile.



2. It is easy to operate with a short rope and a calf leg.



3. The material is soft and smooth, and will not cause damage to the calf.



Hailian Packaging Equipment Co.,Ltd

☎ 0086 13301569240

✉ sales2@dairymachinery.cc

🌐 mobilemilkingmachine.com

Nanzhuang Village, Economic Development Zone, Yangshe Twon, Zhangjiagang City, China