

SS Material Cattle Fluid Feeder Dairy Machinery Appliance For Illness Cow Dairy Farm

Our Product Introduction

for more products please visit us on mobilemilkingmachine.com

Basic Information

- Place of Origin: Jiangsu
- Brand Name: chuangu
- Certification: CE ISO FDA SGS
- Model Number: HL-Q1
- Minimum Order Quantity: 1
- Price: USD\$250/set
- Packaging Details: Carton
- Delivery Time: 5-8days
- Payment Terms: T/T, Western Union,Paypal
- Supply Ability: 100set/month



Product Specification

- Material: Stainless Steel
- Package: Carton
- Usage: Cow Feeding
- Model: HL-Q1
- MOQ: 1 SET
- Delivery Time: 2-3days
- Highlight: cattle hoof trimming discs, cattle nose ring



More Images



Product Description

SS Material Cattle Fluid Feeder Dairy Machinery Appliance For Illness Cow Dairy Farm

Introduction

Product name: Cattle Fluid Feeder

Product number: HL-Q1

Product introduction: Cattle Fluid Feeder is a three-part instrument consisting of a drug pumping device, a transparent catheter and a stomach tube, where in the drug pumping device is connected to the stomach tube through a transparent catheter, the stomach tube is inserted into the rumen of the cow through the esophagus, and then the medicine pumping device sucks. The liquid medicine is delivered to the stomach tube and enters the rumen to complete the hydration of dehydrated cattle.

Model	HL-Q1
Material	Stainless Steel
Package	Carton
Usage	Cow Feeding

使用说明

Directions
for use

创普机械 Chuang Pu

步骤一

Step 1

Prepare the filling solution



According to the appropriate proportion, flush the liquid with warm water and pour it into the bucket.

步骤二

Step 2

Prepare the cows waiting to be fed



Keep the bull's head low and reach forward so that the oral esophagus is basically in a straight line through the rumen.



Attach the esophageal guide tube connected with the bull nose clamp carefully to the oral cavity.



Using bull nose clamp to grip the cows' nose.

步骤三 Step 3

Insert esophageal irrigation tube correctly

创普机械

Chuang Pu



Carefully insert the esophageal irrigation tube. If you feel that there is even resistance in the mouth of the cow, stop for a while and allow the cow to do his own swallowing and swallow the tube. Don't push it.

步骤四 Step 4

Inject the liquid into the rumen




Put the pumping device into the bucket, press the pumping device by hand to pump the liquid medicine to the stomach tube, enter the rumen, and complete the hydration of dehydrated cattle.

产品细节 Product details

Product details

创普机械 Chuang Pu

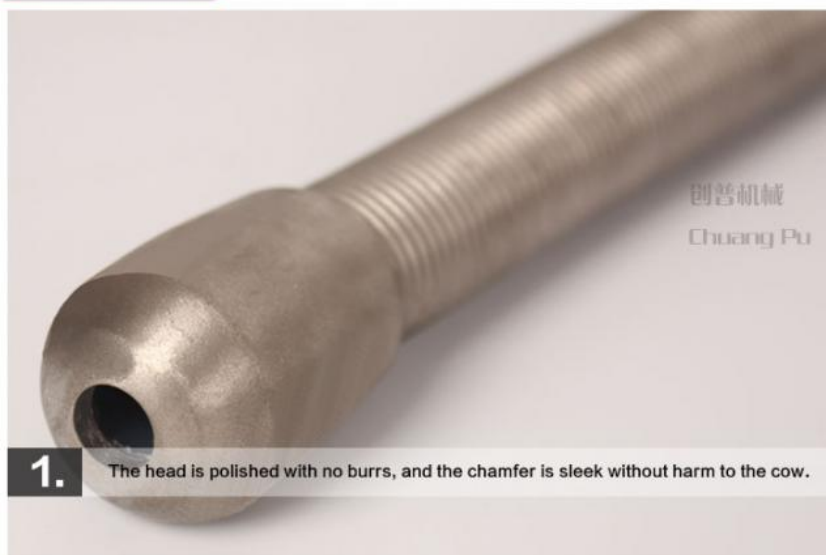

创普机械

Chuang Pu

1.

The head is polished with no burrs, and the chamfer is sleek without harm to the cow.



2. Nitrile rubber seals are non-toxic, safe and durable.



3. Manually pressed air pump switch, which is easy to operate.



Hailian Packaging Equipment Co.,Ltd

☎ 0086 13301569240

✉ sales2@dairymachinery.cc

🌐 mobilemilkingmachine.com

Nanzhuang Village, Economic Development Zone, Yangshe Twon, Zhangjiagang City, China