



# Electric 220v 50hz Cow Dehorner With Imported LEIYA Grinder

## **Basic Information**

Place of Origin: China
Brand Name: Chuangpu
Certification: ISO
Model Number: HL-Q15A
Minimum Order Quantity: 1 Set
Price: \$550/Set
Packaging Details: Carton

Delivery Time: 5-8 working days
 Payment Terms: T/T, Western Union
 Supply Ability: 30 sets for each month



## **Product Specification**

Name: Cow DehornerVoltage: 220V 50Hz

• Function: Remove The Horns From (an Animal)

• Grinder: Leiya

Blade Life: For 1500 Cows

• Weight: 11KG

• Highlight: 50hz cow dehorner,

imported leiya grinder cow dehorner, 1500 blade life cow dehorner



# More Images







## **Product Description**

## HI-Q15A Electric Cow Dehorner From China Chuangpu With Imported LEIYA Grinder

#### **Products Introduction:**

The cow dehorner is to cut the angle ring from the bottom of the horn with a electric cutter after holding cow, and the angular speed is fast and

easy to operate. Dehorning can prevent cows from being injured by fighting, especially the top of the cow that is not followed by the breast.

The dehorned cow is quiet and easy to manage.

#### **How to Use Cow Dehorner?**

- 1. Fix the horns, plug in the power supply, wait for the saw blade to heat up, push the saw blade, and then go to the corner at the moment of pressing. The cow can hardly feel the pain.
- 2. The saw blade is sharp and the surrounding design is precise, which protects people and animals.
- 3. Side orientation with removable grip for backhand operation
- 4. User-friendly design, switch at the handle, easy to use.

#### Attention:

### Attention!

- 1.As to young cow (not pregnant or in second trimester) and dairy cows in the late milking period, to avoid causing miscarriage and
- a sharp drop in milk production, the dehorning time should be selected in the seasons with less mosquitoes and cool weather to prevent infection.
- 2. During the process of cutting horn, the saw blade rotates at high speed and produces extremely high temperature, and then the capillaries in

the horn have been burnt, so this method has less bleeding and faster speed. If there are bleeding capillaries, it can use an electric iron to stop

bleeding, and then anti-inflammatory drugs are used to disinfect the wound.


3. The horn cavity is filled with sterilized cotton after removing horn, remember not to plug it too tightly. Generally, the wound can be naturally flat

for about two months. Please observe the condition of cow after dehorning, and deal with the infection in time.







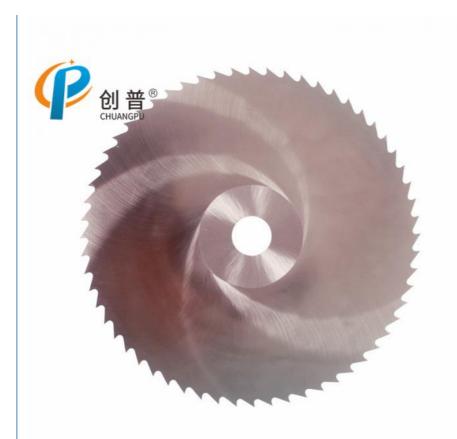








# Hailian Packaging Equipment Co.,Ltd



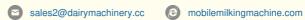



Nanzhuang Village, Economic Development Zone, Yangshe Twon, Zhangjiagang City, China