

## Stainless Steel Bloodless Burdizzo Castrator Veterinary Instruments For Cattle

### Our Product Introduction

#### Basic Information

- Place of Origin: Jiangsu
- Brand Name: chuangupu
- Certification: CE ISO FDA SGS
- Model Number: HL-Q2C
- Minimum Order Quantity: 1
- Price: USD\$30/set
- Packaging Details: Carton
- Delivery Time: 5-8days
- Payment Terms: T/T, Western Union,Paypal
- Supply Ability: 100set/month



#### Product Specification

- Material: Stainless Steel
- Package: Carton
- Product Name: Sheep Castrator
- Feature: Easy Operation
- Keyword: Veterinary Bloodless Castrator Instrument
- Delivery Time: 2-3days
- Highlight: **Bloodless Veterinary Castrator,  
Easy Operation Burdizzo Castrator,  
Bloodless Burdizzo Castrator**

## Product Description

### Agriculture Stainless Steel Bloodless Castrator Cultivation For Farm

#### Product Introduction:

Articulated with closing stop. Made of high quality forged steel. Supplied with a tightening key. The castration can be effected as soon as the testicles have reached their final firmness (not their final size). The bloodless castrator crushes the testicular cord through the scrotum skin.

#### Usage instructions :

- identify the testicular cord -block it with the castrator for 5 minutes
- repeat the same operation 1-2 cm (1/2" - 3-4") lower on the same cord. The non-crushed space maintain the irrigation and prevents from gangrene.
- execute these operations on the second testicular cord
- never try to crush simultaneously both testicular cord. It may be ineffective and it will damage the castrators.

Model	HL-Q2C
Material	Stainless Steel
Package	Carton
Usage	Sheep Castrator
Animal	Sheep or Goat
Weight	1.14KG
Country	China
Delivery time	3-5 days

#### Using bloodless castration method:

livestock binding (veterinary terminology, the reliable and stable control of livestock), after the local anesthesia, the operator grasps the scrotum neck of livestock (such as cattle, sheep, etc.) by hand, and squeezes the testicles to the bottom of the scrotum, the spermatic cord is pushed to the outside of the neck of the scrotum, and is clamped on the inner skin of the spermatic cord with a long-handed sperm stringing clamp to prevent the spermatic cord from sliding and then open the jaw of the bloodless castration clamp and clamp it in the long handle. The ejaculation fixed clamp is fixed at 3-5CM above the fixed point, and the assistant slowly closes the handle. The clamp point should be at least 1 cm above the testicle. After confirming that the spermatic cord has been clamped by the jaws, forcefully close the gripper handle and you will hear a crisp click" sound, indicating that the spermatic cord has been chipped. After the clamp handle is closed, it should be kept for at least one minute, then loosen the jaws to ensure that the spermatic cord has been broken, loosen the forceps, and clamp the second time on the spermatic cord at 1.5-2.0CM below it to ensure the surgical effect.

#### Bloodless castrator for goat





**Hailian Packaging Equipment Co.,Ltd**

☎ 0086 13301569240

✉ sales2@dairymachinery.cc

🌐 mobilemilkingmachine.com

Nanzhuang Village, Economic Development Zone, Yangshe Twon, Zhangjiagang City, China