

Elastrator Plier Applicator Bull Bloodless Castrator Ring Rubber

Our Product Introduction

for more products please visit us on mobilemilkingmachine.com

Basic Information

- Place of Origin: Jiangsu China(Mainland)
- Brand Name: CHUANGPU
- Certification: ISO
- Model Number: HL-Q2A Q2, Q2B ,Q2C
- Minimum Order Quantity: 1set
- Price: USD\$95.00/set
- Packaging Details: CARTON
- Payment Terms: Western Union, MoneyGram, T/T, L/C
- Supply Ability: 100SET/MONTH



Product Specification

- Features: Simple Operation Easy To Use Bloodless Castration
- Material: Stainless Steel
- Application: Horse, Cow, Goat, Pig
- Condition: New
- Highlight: **Elastrator Bloodless Castrator,
Simple operation Bloodless Castrator,
Simple operation bull castrator**

Product Description

Elastrator Plier Applicator Bloodless Castrator Ring Rubber

High-quality plastic steel material.

These pliers are designed for animal castration and tail-cutting without harming them and bleeding.

Suitable for pig,beef,sheep,Use rubber bands in conjunction,

Broken animal tails,

And broken calf horns,

Balloon mouth expansion etc.

Professional castration tool for animals,it is no harm to animals.

Easy to operate and convenient to handle after operation.

Made of stainless steel and High-strength Plastic steel ,it is high intensity and durable.

Strong expansion power and great to use.

Ring is made of rubber,it is high elastic and can be reusable.

Model	HL-Q2	HL-Q2A	HL-Q2B	HL-Q2C
Name	Bloodless Castrating(Burdizzo)			
Material	Stainless steel			
Animal	Cow	Horse, Deer, Mule	Pig / Buck	Goat
Weight	2.18kg	1kg	0.65kg	1.8kg



产品简介

Product introduction

创普机械 Chuang Pu

Product name: Bloodless Castrator

Product introduction: The bloodless castrator is a scientific veterinary surgical instrument. It is suitable for castration of livestock such as bulls, goat and pigs.

The device achieves the purpose of surgery by clamping the animal spermatic cord through the scrotum of the livestock, and does not need to incision on the scrotum of the livestock, so it is called "no blood castration".

Bloodless Castrator for cow




Bloodless castrator for goat



Bloodless castrator for pig




Bloodless castrator for horse, deer and Mule

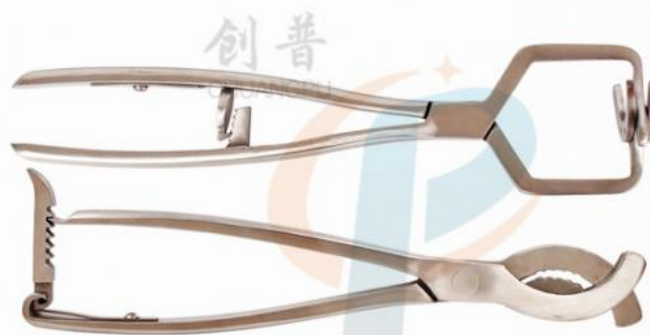

Using bloodless castration method: livestock binding (veterinary terminology, the reliable and stable control of livestock), after the local anesthesia, the operator grasps the scrotum neck of livestock (such as cattle, sheep, etc.) by hand, and squeezes the testicles to the bottom of the scrotum, the spermatic cord is pushed to the outside of the neck of the scrotum, and is clamped on the inner skin of the spermatic cord with a long-handled sperm stringing clamp to prevent the spermatic cord from sliding and then open the jaw of the bloodless castration clamp and clamp it in the long handle. The ejaculation fixed clamp is fixed at 3-5CM above the fixed point, and the assistant slowly closes the handle. The clamp point should be at least 1 cm above the testicle.



After confirming that the spermatic cord has been clamped by the jaws, forcefully close the gripper handle and you will hear a crisp "click" sound, indicating that the spermatic cord has been chipped. After the clamp handle is closed, it should be kept for at least one minute, then loosen the jaws to ensure that the spermatic cord has been broken, loosen the forceps, and clamp the second time on the spermatic cord at 1.5-2.0CM below it to ensure the surgical effect.

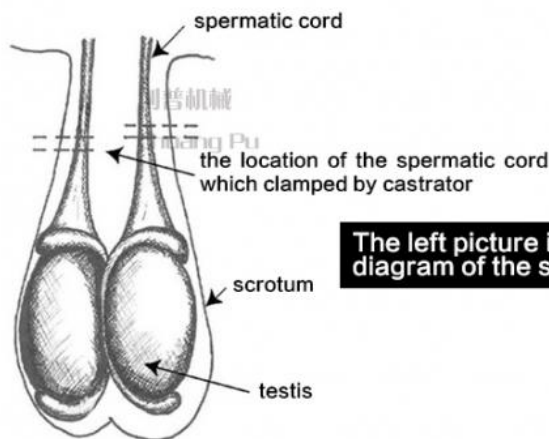

The left picture is a schematic diagram of the surgical procedure



1. The gap between the jaws is tight, the success rate of the operation is high.



2. Selected high quality stainless steel, excellent workmanship.



3. Non-slip design, easy to use (the end of the handle is anti-slip sleeve).



Hailian Packaging Equipment Co.,Ltd

☎ 0086 13301569240

✉ sales2@dairymachinery.cc

🌐 mobilemilkingmachine.com

Nanzhuang Village, Economic Development Zone, Yangshe Twon, Zhangjiagang City, China